

	Medium from Which Anaerobes were Recovered		
	Direct Plating	Enrichment Broth	AnaSelect Broth
1	Peptostreptococcus spp. Viridans strep	Viridans Strep	Viridans Strep
2	Peptostreptococcus spp., Prevotella spp.	Coag Neg Staph	Peptostreptococcus spp. Prevotella spp.
3	Peptostreptococcus spp. Viridans strep	Viridans Strep	Peptostreptococcus spp.
4	Peptostreptococcus spp. S. aureus	S. aureus	S. aureus
5	Peptostreptococcus spp. Coag Neg Staph	Coag Neg Staph	Peptostreptococcus spp. Coag Neg Staph
6	No Growth	Peptostreptococcus spp.	Peptostreptococcus spp.
7	S. aureus	S. aureus	S. Intermedius S. aureus
8	B. fragilis group S. aureus Diphtheroids	B. fragilis S. aureus Diphtheroids	B. fragilis S. aureus
9	Peptostreptococcus spp.	Peptostreptococcus spp.	Peptostreptococcus spp.
10	Ps. aeruginosa	Ps. aeruginosa	Propionibacterium
11	Peptostreptococcus spp. Coag Neg Staph Diphtheroids	Peptostreptococcus spp. Coag Neg Staph Diphtheroids	Peptostreptococcus spp. Coag Neg Staph Diphtheroids
12	S. aureus Coag Neg Staph Diphtheroids	S. aureus Coag Neg Staph Diphtheroids	S. aureus Peptostreptococcus spp.
13	S. aureus	S. aureus	Str. intermedius
14	B. fragilis group Clostridium spp.	B. fragilis group Clostridium spp.	B. fragilis group Clostridium spp.
15	S. aureus Diphtheroids	S. aureus Diphtheroids	S. intermedius
16	S. intermedius Propionibacterium spp. B. fragilis group	S. intermedius Propionibacterium spp. B. fragilis group	S. intermedius Propionibacterium spp. B. fragilis group
17	Bacteroides spp. Fusobacterium spp. Group B Strep Diphtheroids Coag Neg Staph	Bacteroides spp. Fusobacterium spp. Group B Strep Diphtheroids Coag Neg Staph	Bacteroids spp. Fusobacterium spp. Coag Neg Staph
18	B. fragilis group	B. fragilis group	B. fragilis group
19	No Growth	B. fragilis group S. intermedius Coag Neg Staph	B. fragilis group S. intermedius Coag Neg Staph
20	Peptostreptococcus spp. S. aureus Coag Neg Staph	Peptostreptococcus spp. S. aureus Coag Neg Staph	Peptostreptococcus spp. S. aureus Coag Neg Staph
21	Pseudomonas spp.	Pseudomonas spp.	C. perfringens
22	A. baumannii S. aureus	A. baumannii S. aureus	Peptostreptococcus prevotii